

Basic instructions for AR-200

Ceiling Hoist



Charging

- You should continuously charge the battery when the ceiling hoist is not in use. Never allow the battery to fully discharge.
- The battery is charged using the wall holder:
 1. Place the hand control in the wall holder.
 2. Check whether the charging indicator light is lit.



Battery status


The lighting on the underside of the hoist shows the status of the batteries:



Green: batteries are charged



Orange: batteries should be charged at the earliest opportunity

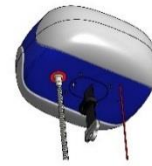


Red: the lift should no longer be used and should immediately be charged!

Emergency stop and emergency lowering function

- Pull the red emergency stop cord to immediately stop the lift.
- Continue to pull the cord to slowly lower the yoke. This downwards motion will stop as soon as you release the cord.

The AR-200 will then be fully switched off.

- To reactivate the AR-200, press the blue button on the hand control.

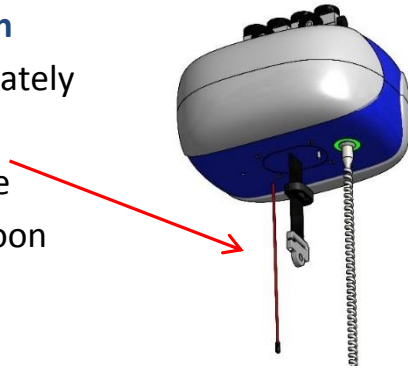

Hand control



The hand control has the following functions:

- Move upwards and downwards
- Bring to a seated or sitting position (electrical 4-point tilt yoke)
- Switch on and off
- Indicator light shows whether the battery is charged.